

Canine Progesterone Dried Spot (DS) Collection Kit



Ensure Accurate Timing For Successful Breeding

Dog breeding requires hard work and dedication. Progesterone testing is a valuable tool for breeders and veterinarians to know when a female dog is ovulating, and to determine the optimal time to breed. The Seroclinix **Canine Progesterone Test** is an accurate and convenient method to monitor levels of progesterone (P4) during the estrous cycle.

Why Choose Seroclinix Lab Service?



We use the latest technology to ensure reliable results

Seroclinix understands the importance of accurate results to help maximize fertility and the chances of conception. Testing is conducted on a Liquid Chromatography Mass Spectrometry (LC-MS), the gold standard used in clinical reference labs to achieve quantitative results with the highest degree of accuracy and precision compared with more traditional technologies.

Simplified Sample Collection And Handling For Fast, Accurate Results

Our Dried Spot (DS) Collection Kits provide an easy, cost-effective sample solution to collect blood or serum samples in a veterinarian's office or on-farm in remote locations where veterinary facilities are not readily available. The sample is dried at room temperature and then shipped to one of our lab in an envelope by courier or parcel post. Multiple tests can be run from each sample.



"Where Advanced Human Diagnostics Meets The Animal World . . . Because Our Animals Deserve The Best"

Canine Progesterone Dried Spot (DS) Collection Kit

Using our lab service

Order a Dried Spot (DS) collection kit on-line at www.seroclinix.com

Your kit will include materials to collect, prepare and ship your blood sample to our lab. **Note: The cost of the kit includes the subsequent lab test.**

Collect the blood sample and prepare the DS Collection Card

Apply a technique that is quick and appropriate to draw a small blood sample (at least 0.5mL). Transfer blood from the syringe to an EDTA tube and then centrifuge. Carefully dispense serum and spot onto the surface of the collection card and let the card dry at room temperature.

For the best method to draw a blood sample and instructions for Dried Spot collection cards refer to our videos at www.seroclinix.com

Send us the DS Collection Card for testing.

- ✓ Log-in to our lab service portal at www.seroclinix.com
- ✓ Complete the on-line requisition for the test
- ✓ Follow the instructions in the kit to package and ship the DS Collection Card
- ✓ Ship your sample economically via your preferred courier or express mail

No costly, bulky packaging or ice packs.

Access your results.

Once your sample arrives at the Seroclinix lab, results will be e-mailed within 24-48hrs.

Ordering Info

Cat# SER-1100

Canine Progesterone Dried Spot (DS) Collection Kit

5 Tests/Kit

Each kit includes materials to collect, prepare and ship 5 serum samples to our lab. The cost of the kit includes the subsequent lab tests for Progesterone.

- To learn more about our lab services and our complete test menu contact: customerservice@seroclinix.com



Corporate Office & Lab:

5700 Timberlea Blvd, Unit 1
Mississauga Ontario Canada L4W 5B9
P 647-479-2944 | F 905-212-4699 | Toll Free 1.877.453.0537
www.seroclinix.com

Satellite Lab:

170 Northpointe Parkway, Suite 500
Amherst, NY USA 14228
Toll Free 1-877-453-0537 | F 905-212-4699

